

# ZIPZOC™

## Zinc Oxide impregnated stocking

**NOW**  
Available from  
Evolan Pharma

ZIPZOC is a sterile zinc oxide impregnated stocking, which is indicated as an aid to the treatment of chronic leg ulceration and other associated skin conditions. Where chronic venous insufficiency exists, ZIPZOC can be used as a primary contact layer under compression bandaging or hosiery kits. ZIPZOC is a preservative free, single use dressing which means there may be a reduced risk of skin reaction from sensitivity. ZIPZOC contains white soft paraffin and liquid paraffin so it won't dry out or adhere to the wound if changed sufficiently frequently. ZIPZOC works by forming a barrier on the surface of the skin which protects against moisture and irritants

### Indications

Use as an aid to the treatment of chronic leg ulcers and other associated skin conditions.

### Ease of use

ZIPZOC's stocking presentation means it is easy to apply and remove, therefore saving time during dressing changes.

- Simple and easy to apply
- Soothing and comfortable
- Sterile
- Low adherent
- Preservative free



ZipZoc  
20% Zinc Oxide

### Ordering Information

768343 66151540 ZIPZOC 10 5060452010021  
768344 66151550 ZIPZOC 4 5060452010038

Order online at [www.uniphar.ie](http://www.uniphar.ie) through the DOE/PIMS Dispensary System  
Jonathan Brady RNID RSCN 083 862 8343 [jonathan.brady@evolan.se](mailto:jonathan.brady@evolan.se)

 uniphar  
medical

 EVOLAN.

Evolan Pharma AB  
Box 120, 182 12 Danderyd, SWEDEN  
E [pastebandages@evolan.se](mailto:pastebandages@evolan.se) T +46-8-544 960 30  
[pastebandagesevolan.com](http://pastebandagesevolan.com)

1. Williams C. Examining the range of medicated and paste-impregnated bandages. British Journal Of Nursing 1999; 8(15): 119-120

**ZIPZOC** (zinc impregnated medicated stocking) **Abbreviated prescribing information** Refer to summary of product characteristics before prescribing. **Composition:** A sterile rayon stocking impregnated with an ointment containing 20% Zinc Oxide. Each stocking (80cm x 7cm) contains about 41.5g of ointment. **Therapeutic indications:** Aid to the treatment of chronic leg ulcers and other associated skin conditions. Where chronic venous insufficiency exists, the medicated stocking can be used as a primary contact layer under compression bandaging or hosiery. **Method of Administration:** Because ZIPZOC is preservative free, it must be changed no less frequently than weekly. For topical application, ZIPZOC should be applied to cover the lower leg from the base of the toes to below the knee. All folds should be smoothed out. To protect clothing, a suitable outer bandage should be worn. If chronic venous insufficiency exists, ZIPZOC may be used as a primary contact layer under compression therapy and may be left in situ for one week, before re-application. **Contra-indications:** Arterial leg ulcers. Known hypersensitivity to zinc oxide or the excipients in the ointment. **Special Warnings and Special Precautions:** None known. **Interactions:** None known. **Pregnancy and Lactation:** There is no evidence to suggest ZIPZOC should not be used during pregnancy. **Undesirable Effects:** The skin of leg ulcer patients is easily sensitised to some topical medications. ZIPZOC contains no preservatives thereby reducing the risk of skin reactions. Reported reactions include, rash, erythema, itching and maceration of the wound edge. If the outer bandage is inappropriate or the medicated stocking is not changed with sufficient frequency it can cause the stocking to dry out. **Excipients:** Liquid Paraffin Ph. Eur. White Soft Paraffin BP. **Incompatibilities:** None reported. **Shelf Life:** 3 years. **Storage:** At or below 30°C. Keep all medicines out of the reach of children. **Packaging:** Polyethylene aluminium foil laminated pouches, each pouch containing a single medicated stocking. The outer carton will contain either 4 or 10 pouches. **Legal Category:** P **Market authorization number:** PL44616/0002 **Date of first authorisation/renewal of the authorisation:** 01 October 2006 **Date of revision of text:** 26 October 2018 **Trade Mark of Evolan Pharma AB** **Date of preparation:** October 2018